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Aprobación:

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Asuntos regulatorios

gb ALL-IN-ONE PROTEIN REMOVER SOLUTION FOR ALL TYPES OF SOFT CONTACT LENSES (including new-generation silicon hydrogel lenses)
CLEANING - DISINFECTING RINSING - STORING - WETTING PROTEIN REMOVER
* No rubbing needed for disposable lenses (≤30 days)

INSTRUCTIONS FOR USE

- Before handling your lenses, wash your hands thoroughly with soap and water.
- Remove the lens from your eye, place it on the palm of your hand with a few drops of ALL CLEAN SOFT, and rub* both sides for a few seconds.
- Rinse both sides of the lens for at least 5 seconds using ALL CLEAN SOFT.
- Place case, and cover them both completely with ALL CLEAN SOFT solution. Close the case and shake it gently for a few seconds.
- Keep the lenses in the solution for at least 4 hours (or overnight) to ensure complete disinfection.
- Once you have finished the process, you can place the lens directly into your eye, or rinse it first if you prefer.
- Rinse the lens case with ALL CLEAN SOFT and allow it to air dry.

Last revision date: 2009/11

PRECAUTIONS

- Once opened, the bottle should not be used after 90 days.
- Maintain the bottle well closed after use.
- Do not use the product if the safety seal is missing or broken.
- You may store your lenses up to 30 days in the closed lenscase.
- Do not ingest the solution and keep away from children's reach.
- Keep at room temperature.
- Do not allow the dropper to come into contact with any surface to avoid contamination.
- In case of irritation, consult your contact lens specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.
- Never use tap water to rinse your lenses or lens case.
- Do not use after the expiration date.
- Discard the solution in the lenscase after each use.
- Your specialist will be able to recommend additional products or changes in cleaning procedures in line with your tear evaluation and the kind of lenses you use and their replacement frequency. Always follow your professional's instructions and never change them without first consulting him/her.

COMPOSITION
Buffered isotonic aqueous solution with poloxamer, EDTA, PVP, polyhexanide 0,0002% and protein remover agent.

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