Modelo: Novoxy One Step Bio
REF Troquel: TRO-PROS-005
Dimensiones: 264 x 200 mm








Código: PROS538.2

Colores:

P.432

PROS538.2

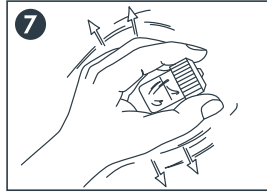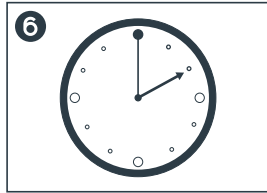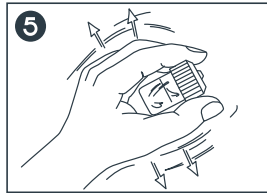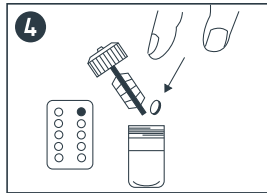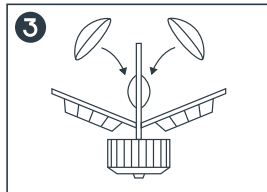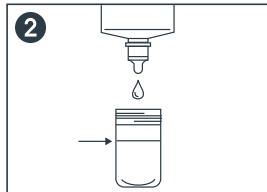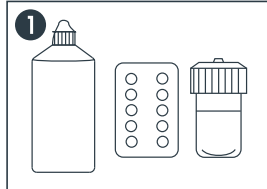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AVIZOR S.A.
La cañada 13-17
28850 Torrejón de Ardoz, Madrid (SPAIN)

Novoxy One Step Bio is a very modern, simple and effective system for daily contact lens care.

By combining the disinfecting power of hydrogen peroxide enhanced by the simultaneous action of a carefully-formulated tablet, the cleaning effect is greatly increased (and selectively targets protein deposits) and the peroxide is neutralised slowly and completely to give a suitable saline solution: • Neutral pH • Isotonic with tear
• No preservatives. Accordingly, lenses are kept perfectly clean, disinfected, hydrated and ready for use maintaining their original parameters. The **Novoxy One Step Bio** system contains: • Bottle of solution, • Blister pack of tablets, • Lens case with elastic valve.

INSTRUCTIONS FOR USE: • Fill the lens case up to the mark with **Novoxy One Step Bio** Solution. • Place the lenses in their respective lens holders. • Add one tablet in the lens case and screw the lid on. • Shake the lens case for a few seconds. **Never use the solution without the neutralizing tablet.** • Wait at least 2 hours for the cleaning, disinfection and neutralisation process to conclude. • The lenses can be used after 2 hours or can be left in the lens case overnight. • The neutralising tablet contains chlorophyll. When the process is finished, the solution where the lenses are should be light green, which indicates that the tablet has worked. • Shake the lens case for a few seconds. • Before handling the lenses to fit them, wash your hands and rinse well to eliminate all traces of soap. • Empty the solution out of the lens case. Keep the empty lens case closed when not in use.
Last revision date: 11/2009

PRECAUTIONS: • Take all hygiene precautions before handling your contact lenses (for example, wash your hands). • Avoid prolonged contact of the initial solution with your skin. If necessary, rinse with tap water. • Do not use the product if the safety seal is missing or broken. • Do not ingest the solution and keep out of children's reach. • Once opened, the bottle should not be used after 60 days. • Maintain the bottle well closed after use. • Once the cleaning and disinfection process is completed, the lenses are in a sterile solution **WHICH CONTAINS NO PRESERVATIVES**. • If the lenses are not used on a daily basis, the green colour of the solution will fade after a few days. Because it does not contain preservatives, if the solution is colourless, this is a sign that you should disinfect the lenses again before wearing them. • If you do not use your lenses every day, it is advisable to replace the cleaning solution in the lens case with a solution containing preservatives, for example **Avizor Saline Solution**, to avoid possible contamination after the disinfection process has concluded. • **Avizor Saline Solution**, may also be used for rinsing your lenses (if they fall, or are temporarily removed, etc...), instead of carrying out a full disinfection with **Novoxy One Step Bio**. • If, exceptionally, the lens wearer's physiology leads to considerable protein deposits that **Novoxy One Step Bio** is unable to remove, seek the advice of your contact lens specialist regarding the advisability of additional protein removal (**Avizor Enzyme** is suitable for this purpose). • Discard the solution in the lens case after each use. • Never use tap water to rinse your lenses or lens case. • Keep at room temperature. • In case of irritation, consult your contact lens specialist. • Before using any ophthalmic medicine, inform your doctor you are a contact lens user. • Do not use after the expiration date. • Do not use if you are allergic to any of the components. • Do not put unneutralised disinfectant solution directly in your eye.

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