# MENICARE PURE

MULTIPURPOSE SOLUTION

## Instructions for use for *MeniCare Pure*

Multipurpose Solution for all rigid gas permeable contact lenses  
This Instructions For Use contains important information.  
Please read carefully and retain for future use.

### INDICATIONS

- DISINFECTING: Effective after 30minutes
- CLEANING
- RINSING
- STORAGE

### TO OPEN

The dropper tip is sealed. To open, screw the cap clockwise as far as it will go. The spike inside the cap will pierce the seal.

### DIRECTIONS

Wash your hands thoroughly before handling your contact lenses:

- Rub the lens carefully with several drops of *MeniCare Pure* for 20 seconds.
- Place the lenses in the right and left lens holder and rinse them with *MeniCare Pure*.
- Fill the lens case with *MeniCare Pure* and soak the lenses for at least 30 minutes or overnight.
- Before insertion, rinse the lenses with *MeniCare Pure*.
- Keep the lens case clean and dry after use.

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 30 days. If you store your lenses for longer periods of time, they must be cleaned and disinfected with *MeniCare Pure* solution every 30 days.

### WARNINGS

- Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.
- Consult your doctor or eye care professional before using any eye medication with contact lenses.

### CONTRA-INDICATIONS

- Do not use in case you are allergic to one of the ingredients.

### PACKAGING

*MeniCare Pure* is available in 250mL and 70mL bottles, including a disposable lens case.

### PRECAUTIONS

- Once opened, use *MeniCare Pure* within 3 months and make sure that the bottle is tightly closed after each use.
- Do not touch the bottle top with your hands.
- Do not swallow.
- Keep out of reach of children.
- Do not use after expiry date.
- Do not use if the film around the cap is broken before first use.
- Do not use with soft contact lenses.
- Keep the lens case clean and dry after use.
- Store at room temperature. (±25°C)
- Store out of sunlight.
- Replace the lens case with every new bottle of *MeniCare Pure*.
- Consult your eye care professional before changing solutions to a different solution.
- Do not use as lubricating and rewetting solution.

### ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult with your eye care professional.

### CONTENTS

- E-Polylysine 0,01% as preservative, Edetate disodium 0,05% as chelating agent, HCO-60 0,1% as surfactant.

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© 2016, Menicon Co., Ltd.