WÖHLK CONTACTLINSEN:

- wöhlk PAUL MAS / wöhlk S / wöhlk AS / wöhlk AS 2 / wöhlk KE / wöhlk BIFO / wöhlk MULTI (FEN)
- CONTACT INDIVIDUAL BIO / CONTACT INDIVIDUAL SH / CONTACT INDIVIDUAL MULTI BIO
- CONTACT INDIVIDUAL MULTI SH / wöhlk Weflex 55 / wöhlk Geaflex 70

CH REP CONTOPHARMA AG,
Eichzün 7,
3800 Interlaken / Switzerland
E-Mail: info@contopharma.ch

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INSTRUCTIONS FOR USE

GENERAL INFORMATION

These instructions for use contain important information for you. Read them carefully and retain them. Contact lenses are medical products and require qualified fitting by an optician, specialist optician or optometrist.

INTENDED USE / REPLACEMENT PERIOD

Reusable hard contact lenses for correction of vision defects (myopia, hyperopia, astigmatism, presbyopia) for daily wear in healthy eyes. **The replacement period** is a maximum of 12 months for the materials CFA, ASF 60, A 100 UV, ASF 120, Boston ES, Boston EQ and Boston XO. **Reusable soft contact lenses** for correction of vision defects (myopia, hyperopia, astigmatism, presbyopia) for daily wear in healthy eyes. **The replacement period** is a maximum of 6 months for the materials Vitaflon B (BIO) and Aerofilcon B (SH) or a maximum of 12 months for the materials Weflicon A and Geafilcon A.

CONTRAINDICATIONS

In general, the product may only be used in conjunction with an intact cornea, healthy eyelids and a healthy conjunctiva unless the eye care professional expressly recommends its use. Contraindications are: • Inflammatory disorders and infection of the eye • Altered corneal sensation • Insufficient tear production • Poor tear film quality • General illnesses insofar as these are associated with a weakened immune system • Lid or corneal defects. Only use contact lenses following consultation with your eye care professional.

CONTENTS

Store your hard contact lenses in a buffered solution preserved with 0.0004% PHMB, 0.1% EDTA. Store your soft contact lenses in a phosphate-buffered saline solution with 0.005% Pluronic F127.

USING YOUR CONTACT LENSES

• Upon first use, use only contact lenses from containers with a undamaged tamper-proof ring. The manufacturer only guarantees that the contact lens is delivered in usable condition if the original contact lens containers are sealed. Use the containers for storage of contact lenses only. • Never use contact lenses from damaged original packaging or which have been opened accidentally. • We recommend storing and using the product at room temperature. • If you have different contact lenses, take care not to mix them up. • We recommend having a pair of glasses with your current prescription in addition to your contact lenses. You should always have these available. • Soft contact lenses should not be used again after drying out. • Hard contact lenses should not be subjected to temperatures above 45 degrees Celsius. • Wear them only while awake and remove them from your eyes before going to sleep. • Your eye care professional will tell you how you can continually increase the wearing time for your contact lenses and what the maximum possible daily wearing time is. • The useful life depends on the contact lens material, the parameters, handling, care and the wearing time. • Do not exceed the recommended useful life and do not use any contact lenses which are past their expiry date. • Upon expiry of the useful life, they must be replaced with new contact lenses. • At the end of the recommended useful life, dispose of the contact lenses in general waste. The original packaging should be disposed of in household waste or through the local recycling system.

INSTRUCTIONS FOR HANDLING THE CONTACT LENSES

• The practitioner will teach the user how to handle the contact lenses. • Keep your fingernails short and avoid sharp edges in order to prevent eye injuries or damage to the lenses. • Wash your hands thoroughly with a non-greasy soap before putting the contact lenses in or taking them out and rinse your hands thoroughly before touching the lenses. The contact lenses may become soiled and/or microbiologically contaminated if you do not wash your hands properly. • Only handle the lenses in a well lit, clean environment. • Getting cosmetics (e.g. hair and skincare products, eyeliner, etc.) in the eye or on the contact lenses should be strictly avoided. • Respect the mechanical sensitivity of the contact lenses during handling. • Do not use damaged contact lenses as these may injure or irritate the eye. • Only change your contact lenses and/or the care product in consultation with your eye care professional. • Visit an eye care professional regularly, and at least every six months, for a contact lens check-up (monitoring). • Rubbing your eyes may cause the contact lenses to slip. • If you are unsure whether your contact lenses are still in your eye, consult an eye care professional in order to prevent infections, etc. • Speak to your eye care professional if you have questions about the handling, care and tolerability of your contact lenses. In the event of eye complaints, immediately remove your contact lenses and consult your eye care professional.

PUTTING CONTACT LENSES IN HARD AND SOFT CONTACT LENSES

• Please wash your hands thoroughly first. • Opening a wöhlk pot: turn the plastic lid counterclockwise to open it. Always make sure that the tamper-proof ring on the lens pot is undamaged before opening. • If the tamper-proof ring is damaged, do not put the contact lenses in! Contact your eye care professional. • To remove the contact lens, unscrew the lid from the pot and carefully remove the plastic seal. • For hard contacts, please note that the contact lens holder is integrated into the seal here. • The contact lens can easily be removed from this holder. • For soft contacts, pour the contents (contact lens and fluid) into your palm and carefully allow the fluid to drain off. • Before putting your contact lens in, now carefully rinse it with a suitable solution recommended by your eye care professional and check the contact lens for damage. • To put the contact lens in, place it on the tip of your index finger. • For soft contact lenses, pay attention to the shape. The outer edge must point up. If it is angled slightly outward, then you need to invert the contact lens and rinse it again. • With the other hand, reach over your forehead and pull the upper eyelid up gently with your index finger. The middle finger of the other hand gently pulls the lower eyelid down at the same time. • The contact lens can now be placed centrally on the cornea of the eye. • It is helpful to initially place the lens in the area below the cornea and then move it up. To remove any possible air bubbles under the contact lens, "shift" it slightly with the lower eyelid. If you feel like the contact lens is sitting well, slowly close your eye and open it again. • If the above method for putting lenses in does not work for you, ask your eye care professional for an alternative. •

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If your vision remains blurry a short time after you have put your contact lenses in, it may be that you have mixed up your right and left contact lenses. Remove the contact lenses and have them checked by an eye care professional. • If your vision is blurry after putting your soft contact lenses in, it may be that the contact lens has been put in inside out. Remove the contact lens and check it. • If the contact lens causes a scratchy feeling on your eye, it may be that there is a foreign body irritation on the cornea or the contact lens is damaged. Remove the contact lenses and have them checked by an eye care professional. • If the contact lens causes a burning sensation, remove the contact lens and rinse it again. If the feeling remains, remove the contact lenses and have them checked by an eye care professional.

TAKING CONTACT LENSES OUT HARD CONTACT LENSES

• Please wash your hands thoroughly first. • Bend your head over a mirror placed horizontally. Open your eye wide so that the hard contact lens is no longer covered by the eyelids. • Place a fingertip perpendicular to the outer corner of the eye. The fingernail should not touch the eye while doing this. • Consciously keep your eye open and pull your eyelids tight towards your ear. The lens will pop out of your eye as a result. In order to prevent mix-ups, always place your contact lenses in the correct chamber (R-right/L-left) of your storage container. Clean this thoroughly first. If the above method for removing your lenses does not work for you, ask your eye care professional for another method.

TAKING CONTACT LENSES OUT SOFT CONTACT LENSES

• With your head tilted slightly forward, look slightly up into a hanging mirror and pull the lower eyelid down gently with the middle finger of your left or right hand. • Use your index finger to move the lens down slightly onto the white conjunctiva area of the eye. • Here, you can gently pinch the soft contact lens between your thumb and index finger and remove it from your eye. Avoid contact with fingernails while doing this. • Continually looking slightly upward while doing this makes it particularly easy. In order to prevent mix-ups, always place your contact lenses in the correct chamber (R-right/L-left) of your storage container. Clean this thoroughly first. If the above method for removing your lenses does not work for you, ask your eye care professional for another method.

CONTACT LENS CARE AND STORAGE

• Safe handling of contact lenses requires systematic and reliable care as well as hygienic storage. • Your eye care professional will suggest suitable care products for soft and hard contact lenses and storage containers for care and storage. **wöhlk** recommends: for soft contact lenses: **wöhlk Aqua Safe Plus**, **wöhlk Peroxid**, **wöhlk Lipidreiner**, **wöhlk Enzym** / for hard contact lenses: **wöhlk Aufbewahrung**, **wöhlk Reinigung**, **wöhlk Peroxid**, **wöhlk Lipidreiner**, **wöhlk Enzym**. Clean and disinfect your contact lenses after each use, observing the instructions for use for the appropriate care product. • The storage container must be cleaned after each use and must be replaced when starting a new bottle of care product. • See the instructions for use for the appropriate care products for information about the use and cleaning of the storage container and use of the care product. • Do not use non-sterile water (e.g. tap water) to clean the storage container. Doing so increases the risk of a serious microbial infection. • If the contact lenses are not worn for an extended period of time and/or if the maximum storage time in the storage container is exceeded, they must be cleaned and disinfected again, observing the instructions for use for the appropriate care product, before use.

INTERACTIONS WITH OTHER PRODUCTS

• In order to prevent undesirable interactions between the contact lenses and the care product, use the care products recommended by **wöhlk** and observe the instructions for use of the care product. • Care products for hard contact lenses should not under any circumstances be used for soft contact lenses. • Tell your eye care professional what medicinal products and dietary supplements you use. • For eye drops and medicinal products, the relevant information in the package inserts for these preparations must be observed. • The tolerability of the contact lenses may be affected by environmental factors or the user's health. Factors include, for example: Aggressive vapours, gases, cigarette smoke, essential oils or irritant gases / changes to the tear film owing to illness, pregnancy, childbirth, medicinal products, dietary supplements, general physical condition and / or eye conditions / climatic conditions in the living and working environment. • In the event that chemical or hazardous substances come into contact with the eye, first rinse it with tap water and then remove your contact lenses where possible. Consult an optician immediately. **SIDE EFFECTS DURING INTENDED USE**

Eye complaints may occur even when wearing contact lenses properly. These may become noticeable through one or more of the following indications or symptoms: • Burning or itching of the eye • Excessive tear production • Severe redness • Deterioration of vision • Allergic reactions. If disregarded, these symptoms can result in serious complications such as: corneal oedema, ulcers of the cornea, corneal infiltration / erosion of the corneal epithelium, keratitis (incl. infectious keratitis) / conjunctivitis / potential for accelerated reduction of the endothelial cells / potential for changes in the shape of the cornea. If these type of symptoms occur, you should immediately remove your contact lenses and contact your eye care professional.

WARNING

• Use only your own contact lenses. Wearing someone else's contact lenses can cause problems such as irritation or eye infection. • Avoid the contact lenses coming into contact with contaminated/non-sterile water such as in swimming pools, lakes or hot tubs, tap water, distilled water or saliva. Otherwise, wearing the lenses may result in serious microbial infections (e.g. acanthamoeba infections when swimming and diving) or, in the case of soft lenses, may result in changes to the shape of the contact lens owing to chlorinated water/saltwater. • Some medicinal products or

dietary supplements may, in isolated cases, result in changes in the colour of contact lenses, changes in shape and/or eye irritation. • Extending the wearing time and wearing the contact lens while sleeping can result in neovascularisation and excessive corneal swelling.

Report serious incidents in connection with our product to wöhlk Contactlinsen GmbH and to the competent authority in your country.

SYMBOLS

Read the product information on the front or back of the label. Peel the wraparound label at the corner peeling point to the STOP marking only. The following symbols may be found on the label:

= manufacturer / = date of manufacture (year-month-day) / = medical product / = batch number / = expiry date (year-month-day) / = heat-sterilised / = simple sterile barrier system / = individual patient, can be used multiple times; replacement interval (number of months) see replacement period section / = caution / = observe instructions for use / = Do not use if packaging is damaged. / = Non-sterile / = Swiss representative / = corner peeling point / = Unique Device Identification

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